



By purchasing the Vaping Toolkit you agree the test strips are self-administered and results are **indicators** based on home test administration and collection conditions that are not overseen.

In the event that a test appears to be positive for the presence of a drug after administering the home test, your purchase agreement dictates that any conclusion as to the results be confirmed at a drug test facility.

Since the home tests are not under perfect lab conditions, results are to be considered an indication of the likely presence of a drug. Confirm by a 2nd test at a local drug testing facility.

Best indicator of drug use history is a hair test – A 90-day past history of use

Parents should note that a drug testing facility can take a hair sample (youth must be present for sample to be taken) that offers a 90 day past-use history. This is a good choice in the event a second test is needed to confirm the indicated results of the home test. It is also a good choice if you are certain your child has been abusing drugs or alcohol.

**You assume responsibility for steps to protect the health & well-being of your child.
Visit drugfree.org for information on how to talk to your child.**

SAMHSA 24/7 Free and Confidential Hotline 888-320-4083

Information on use of Vaping Toolkit test Strips

Detection window

Nicotine – 1-3 days after use depending on volume consumed

THC (Marijuana) – 5-10 days after use depending on volume consumed

Fentanyl – 3 days depending on volume consumed

Alcohol – Up to 80 hours depending on volume consumed

How to administer urine test:

1. Note the name of the drug on the test strip package and expiration date
2. Collect urine in a small clear, clean, disposable cup that is at room temperature
3. Remove test strip cap to expose thin test strip. Do not touch the test strip.
4. Collect urine in the cup. Make sure there is enough to submerge to top of strip
5. Submerge test strip only to the top of the strip. Wait 5 min.
6. A negative test – no presence of drug tested for – will show 2 lines
7. A positive test will only show 1 line